

Cath Strap Attachment Instructions

Step 1 – Attach Base Unit directly to the Table

- A. Remove Table Mat.
- B. Snug Base Unit to table crossing Base Unit by placing solid strap under table (Velcro on table top).
- C. Velcro ends cross to meet exactly.



Step 2 – Strap Velcro should meet exactly

- A. Straps positioned with male Velcro strap first.
- B. Female Velcro strap adheres to male the full length & width of straps.
- C. Press down firmly along strap overlap to assure good adherence.



Step 3 – Replace Table Mat

- A. Locate strap at approximate mid-chest level of patient by sliding along the long axis of the table.
- B. **Note: Position to permit free table movement (panning).**



Step 4 – Proper positioning on patient

- A. Wrap patient snugly keeping IV site accessible and removal tab free.

